JuzoPro

## Lumbal Xtec/Lumbal Xtec Light

Lumbalorthesen · Lumbar orthosis

---

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Käyttöohje · Bruksanvisning · Οδηγίες χρήσης · Kullanım talimatları · Upute za uporabu · Instrukcja użycia · Инструкція по експлуатації · Návod na používanie · Navodila za uporabo · تعليمات الاستخدام

---

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## EN

## Intended purpose

Thanks to its compressive knitted fabric, in combination with stabilising rigid elements and optional straps, the JuzoPro Lumbal Xtec/JuzoPro Lumbal Xtec Light orthosis supports the lumbar region of the back, thus relieving it during movement. JuzoPro Lumbal Xtec: The sacral pad provides stabilisation in the end range of motion and, thanks to friction nubs, has a massaging effect on the muscles.

## Here is how to put on your JuzoPro Lumbal Xtec correctly:

1. Fasten the stabilising girdle as described by the specialist staff between the two rear bars on the fleece belt and press together firmly. The tip of the stabilising girdle must point downwards.
2. To make the product easier to put on, please use two fingers to reach into the finger pockets attached to the closure sections.
3. Now place the orthosis around your body and close the touch fastener on the front of the orthosis in the following order: first position the left closure on the abdomen, then place the right closure on top of it while applying tension and attach them to each other.
4. The orthosis must be put on low enough or be pulled down after donning so the lower edge of the orthosis is just above the groin.

The orthosis fits properly when it surrounds the body without applying too much (painful) pressure. You can readjust it at any time.

Caution: Rough, unfiled finger nails or sharp-edged rings etc. may damage the material.

## Here is how to put on your JuzoPro Lumbal Xtec Light correctly:

1. To make the product easier to put on, please use two fingers to reach into the finger pockets attached to the closure sections.
2. Now place the orthosis around your body and close the touch fastener on the front of the orthosis in the following order: First position the left closure on the abdomen, then place the right closure on top of it while applying tension and attach them to each other.
3. The orthosis must be put on low enough or be pulled down after donning so the lower edge of the orthosis is just above the groin.

The orthosis fits properly when it surrounds the body without applying too much (painful) pressure. You can readjust it at any time.

Caution: Rough, unfiled finger nails or sharp-edged rings etc. may damage the material.

### Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the orthosis can only be realised if it is applied correctly. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. The durability of the material may be affected by certain combinations of the above. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the orthosis is not suitable for treating more than one patient. The sewn-in textile label is important for the identification and traceability of the product. We therefore ask you not to remove the label under any circumstances.

### Washing and care instructions

Please observe the care instructions on the textile label attached to your orthosis. Take the rear stabilising girdle out of the orthosis (JuzoPro Lumbal Xtec), close the touch fasteners and wash the orthosis in gentle or delicate cycle at approx. 30 °C. Wash the orthosis separately the first time (possibility of colour bleeding). If washing by hand, rinse the orthosis well and allow it to dry at room temperature. If possible, lay flat. Do not wring it out. We recommend using the mild Juzo Special Detergent. To reduce the drying time, you can place the orthosis on a thick terry towel, roll it up tightly and squeeze it firmly. Do not leave it in the towel and do not place it on a heater or in the sun to dry. The orthosis is not suitable for tumble drying. Please fasten the stabilising girdle back into the orthosis again (JuzoPro Lumbal Xtec) when the orthosis is completely dry. This Juzo product must not be cleaned

chemically. You can wipe the stabilising girdle clean with a damp cloth. The cover of the stabilising girdle can be washed just like the orthosis.

Tip: a laundry net provides additional protection for the knitted fabric. Please do not use fabric softener!

### Material composition

For specific details please refer to the textile label sewn onto your orthosis.

### Storage information and usage duration

The medical retailer or doctor prescribing the orthosis will recommend exactly how long it should be used for. If the product is being worn and washed every day, its maximum usage duration will be 6 months. If the product is used on an irregular basis, the wearing duration will be longer, up to a maximum of 12 months. Based on individual use, product wear may differ. We therefore recommend that the product is checked by a medical specialist in the event of suspected changes. Store the product in a dry place and protect it from sunlight. Orthosis have a maximum shelf life of 42 months. The usage duration of the medical product is printed on the box with an hourglass symbol. Body measurements may change based on the patient's condition and individual circumstances. We therefore recommend that body measurements are checked regularly by a medical specialist.

### Indications

#### JuzoPro Lumbal Xtec:

Chronic low back pain/ligamentoses (ongoing pain in the lumbar region), degenerative spondylolisthesis (degenerative vertebral slippage), myostatic, muscular insufficiency (postural weakness), osteochondrosis/spondylitis/spondylarthrosis (mild joint wear), post-discectomy syndrome (pain syndrome after surgical removal of intervertebral disc tissue), SI joint syndrome.

#### JuzoPro Lumbal Xtec Light:

Chronic low back pain/ligamentoses (ongoing pain in the lumbar region), myostatic, muscular insufficiency (postural weakness), symphysis loosening, post-discectomy syndrome (pain syndrome after surgical removal of intervertebral disc tissue), SI joint syndrome.

### Contraindications

In case of the following syndromes or clinical pictures, the orthosis should only be worn after prior consultation with the treating physician: Skin diseases or skin irritations; Wounds in the treatment area are to be covered with sterile dressings.

Liability for failure to comply with these contraindications cannot be accepted by the Julius Zorn GmbH Group.

### Side effects

There are no known side effects when using this product correctly. However, if you notice any adverse

changes (e.g. irritation of the skin) during the prescribed use, please consult your doctor or medical retailer immediately. If you know that you are allergic to one or more of the product ingredients, please consult your doctor before using the product. If your symptoms worsen during the time you are wearing the product, please take off the product and consult a medical specialist immediately. The manufacturer does not accept any liability for damage/injuries resulting from improper use or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

### Disposal

As a rule, the product is to be destroyed via the conventional disposal route after the end of use. Please observe the applicable local/national regulations.

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Berro AG · Austrasse 7 · 9055 Bühler · Suisse
**UK responsible person: Juzo UK Ltd**