Life

JuzoPro

Malleo

Sprunggelenkorthese · Ankle orthosis

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Bruksanvisning · Bruksanvisning · Käyttöohje · Bruksanvisning · Οδηγίες χρήσης · Kullanım talimatları · Upute za uporabu · Instrukcja użycia · Инструкция по эксплуатации · Návod na používanie · Navodila za uporabo · تعليمات الاستخدام

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EN

Intended purpose

Intended the JuzoPro Malleo stabilises the ankle joint with two rigid padded elements that are joined together in a way that allows for flexibility and adaptability and lie on the sides of the joint. It limits the pronation and supination movement, i.e. the inward or outward twisting.

How to put on the product correctly:

Put on your support while sitting down to prevent injuries or falling.

1. Open the two straps on the orthosis shells, fold open the shells and lay the orthosis down on the floor in front of you with the pads facing up. Now place your heel on the piece connecting the orthosis shells. As you do this your foot should be at a 90° angle to your lower leg. Now fold both shells around your ankle so that your anklebones rest in the center of the corresponding protrusions provided.
2. Now pass the orthosis strap around your leg and attach it to the hook and loop pad on the opposite shell. Continue to wrap the strap around your leg and then attach the hook and loop fastener located on the end of the strap to the strap itself. Repeat the previous step for the lower orthosis strap.

The correct fit of the orthosis is assured when it encloses the ankle tightly but does so without exerting a too strong (painful) pressure. The orthosis can be readjusted at any time by repeating step 2 at any time. To achieve an optimal therapy result you should wear the orthosis over your socks while wearing runners / shoes.

Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the orthosis can only be realised if it is applied correctly. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. The durability of the material may be affected by certain combinations of the above. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the orthosis is not suitable for treating more than one patient. The sewn-in textile label is important for the identification and traceability of the product. We therefore ask you not to remove the label under any circumstances.

Washing and care instructions

Please observe the care instructions on the textile label on your JuzoPro Malleo. Detach the pads from the orthosis shells (hook and loop fastener) and wipe the shells with a moist cloth. Handwash the pads with laundry detergent for delicate fabrics every day or between uses at approx. 30 °C. We recommend using the mild Juzo Special Detergent. Afterwards please rinse out the pads well and allow them to dry in a free-hanging position at room temperature. To reduce the drying time, before hanging up the pads you can place them on a thick terry towel, roll them up tightly and squeeze firmly. Do not leave in the towel and do not place on a heater or in the sun to dry. The pads are not suitable for tumble drying. Please do not reattach the pads to the shells until they are fully dry. This Juzo product must not be cleaned chemically.

Material composition

For precise details, please see the textile label that is sewn into your product.

Storage information and usage duration

Store in a dry place and protect from sunlight. The medical retailer or doctor prescribing the orthosis will advise on exactly how long it can be used for.

Notes for specialist staff

Please note the applicable official regulations relating to the issue and adjustment of orthoses by authorised specialist staff.

Adjustment: JuzoPro Malleo can be adjusted for the particular foot width. To do so detach the pad at the

sole of the foot from the shell. You can now loosen the hook and loop fastener and adjust the distance between the shells as required.

Indications

Non-surgical and postoperative treatment of ligament injuries, chronic ankle instability, following ligament lesions and/or sprains, stabilisation of the ankle during exercise, as substitute for plaster cast

Contraindications

If the following symptoms exist, the orthosis should only be worn after consultation with the physician: lymph drainage disorders, arterial flow impairment, post-thrombotic conditions skin disorders or skin irritations; open wounds in the treated body region should be covered sterilely.

Liability for failure to comply with these contraindications cannot be accepted by the Julius Zorn GmbH Group.

Side effects

There are no known side effects when using this product correctly. However, if you notice any adverse changes (e.g. irritation of the skin) during the prescribed use, please consult your doctor or medical retailer immediately. If you know that you are allergic to one or more of the product ingredients, please consult your doctor before using the product. If your symptoms worsen during the time you are wearing the product, please take off the product and consult a medical specialist immediately. The manufacturer does not accept any liability for damage/injuries resulting from improper use or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

As a rule, the product is to be destroyed via the conventional disposal route after the end of use. Please observe the applicable local/national regulations.

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**Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany
**Berro AG** · Austrasse 7 · 9055 Bühler · Suisse
UK responsible person: Juzo UK Ltd