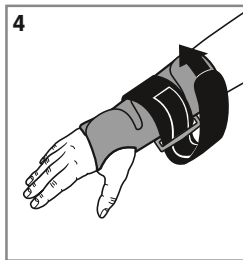
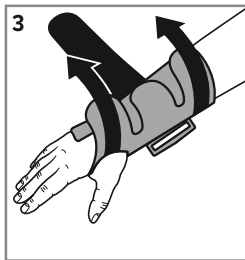
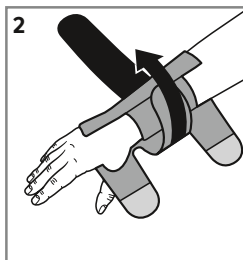
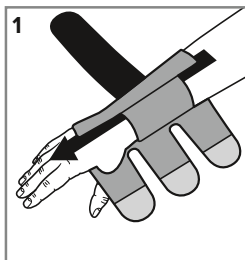
JuzoPro

Manu

Handgelenkorthese · Wrist orthosis

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Bruksanvisning · Bruksanvisning · Käyttöohje · Bruksanvisning · Οδηγίες χρήσης · Kullanım talimatları · Upute za uporabu · Instrukcja użycia · Инструкция по эксплуатации · Návod na používanie · Navodila za uporabo · تعليمات الاستخدام

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EN

Intended purpose

The JuzoPro Manu, made out of inelastic materials with adjustable, partly removable reinforcing elements, immobilises the wrist in at least two movement directions.

This is how to put on your JuzoPro Manu correctly:

1. Open all velcro fasteners and slip your hand into the orthosis so that the velcro fasteners are on top and your thumb protrudes from the opening provided for it.
2. First, fasten the centre velcro, or for the long model fasten the two medium velcros.
3. Then secure the velcro across the hand, and finally secure the fastener on the forearm.
4. Move the circular strap around the wrist and push it through the plastic strap. Then pull it back over it and close the velcro fastener.

Proper fit of your orthosis is ensured if it firmly encloses the wrist without exerting undue (painful) pressure. You can readjust your orthosis any time by repeating steps 2–4.

Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The orthosis can only be fully effective when it has been put on correctly and is being worn in accordance with treatment recommendations. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may negatively impact on the durability of the fabric.

We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the orthosis is not suitable for treating more than one patient. The sewn-in textile label is important for the identification and traceability of the product. We therefore ask you not to remove the label under any circumstances..

Washing and care instructions

Please observe the care instructions on the textile label attached to your orthosis. Please remove the circular strap. Afterwards remove the three aluminum splints from their pockets. Close all touch fasteners and handwash the orthosis and strap with laundry detergent for delicate fabrics every day or between uses at approx. 30 °C. We recommend using the mild Juzo Special Detergent for this. Wash the orthosis separately the first time (possibility of colour bleeding). Thereafter, please rinse the orthosis fabric well and let it dry hanging freely; do not wring it out. To reduce the drying time, you can place the orthosis on a thick terry towel, roll it up tightly and squeeze it firmly. Do not leave it in the towel and do not place it on a heater or in the sun to dry. The orthosis is not suitable for tumble drying. Please return the aluminum splints to their pockets only after the orthosis fabric has dried completely. This Juzo product should not be chemically cleaned.

Material composition

Details are provided on the label sewn into your orthosis.

Storage information and usage duration

Store in a dry place and protect from sunlight. The medical retailer or doctor prescribing the orthosis will advise on exactly how long it can be used for.

Notes for the orthopaedic technician

The three stabilisation splints are anatomically preformed. They may be individually removed from their pockets either for setting, or for specific indications.

Indications

Moderate to severe sprains and strains, tendon / joint injuries, moderate to severe tenosynovitis (inflammation of a tendon and its sheath), carpal tunnel syndrome, rheumatoid arthritis (chronic irritation), osteoarthritis, post-traumatic / postoperative immobilisation (substitute for plaster cast).

Contraindications

If the following symptoms exist, the orthosis should only be worn after consultation with the physician:

lymph drainage disorders, arterial flow impairment, post-traumatic conditions, skin disorders or skin irritations; Wounds in the treatment area are to be covered with sterile dressings.

The Julius Zorn Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your symptoms worsen during the time you are wearing the product, please take off the orthosis and consult a medical specialist immediately. The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse.

In the event of complaints related to this product, such as damage to the material or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

As a rule, the product is to be destroyed via the conventional disposal route after the end of use. Please observe the applicable local/national regulations.

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**Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany · juzo.com
**Berro AG** · Austrasse 7 · 9055 Bühler · Suisse
UK responsible person: Juzo UK Ltd