	DE EN FR IT PL DA	Webseite zur Patienteninformation Patient information website Site Internet d'informations pour les patients Sito web con informazioni per i pazienti Strona internetowa z informacjami dla pacjentów Hjemmeside til patientoplysninger
	DE EN FR IT PL DA	Nach Gebrauch wegwerfen Discard after use Usage unique. Jeter après utilisation Smaltire dopo l'uso Wyrzucić po użyciu Bortskaf efter brug
	DE EN FR IT PL DA	Verwendbar bis Usable until Date d'expiration Utilizzabile fino a Data ważności Kan bruges indtil
	DE EN FR IT PL DA	Hersteller des Produktes Manufacturer of the product Fabricant du produit Produttore dell'articolo Producent produktu Producent
	DE EN FR IT PL DA	Medizinprodukt Medical device Dispositif médical Prodotto medico Urządzenie medyczne Medicinsk produkt
	DE EN FR IT PL DA	Artikelnummer Article number Référence Numero articolo Numer artykułu Varenummer
	DE EN FR IT PL DA	Fertigungslosnummer, Charge Production batch number, batch Numéro de lot de fabrication Numero di lotto di produzione, partita Numer partii produkcyjnej, szarża Batchnummer, batch
	DE EN FR IT PL DA	Pharmazentralnummer Pharmaceutical registration number Numéro central de la pharmacie PZN (Germania) Numer centralny farmacji Farmacentralnummer

SPORLASTIC ORTHOPAEDICS

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INSTRUCTIONS FOR USE EN

INTENDED USE

MALLEODYN® S3 is a removable ankle brace for dynamic mobilisation in a defined position.

INDICATIONS

- Conservative / postoperative
- Acute ankle injuries
- Acute capsular ligament instabilities and ruptures
- Chronic capsular ligament instabilities
- Supination trauma
- Ankle sprains
- All indications where stabilisation or movement restriction of the ankle joint with subsequent functional mobilisation of the injured or damaged structures is necessary

CONTRA-INDICATIONS

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be cared for.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenic disorders of the sensory system and skin trophism in the body area to be cared for (sensory disorders with and without skin damage).

SIDE-EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

PUTTING IT ON AND USE

- We recommend wearing a stocking under the brace.
- Open the figure 8 rein at the rear two velcro ends and the orange click fasteners on the lower leg and foot sections.
- Place your foot, at a 90° angle, in the opened brace **1A**. Position your foot so that the outer ankle is centred in the recess of the side splint **2A**.
- Position the enclosed cooling pad from the inside between the stocking and MALLEODYN® S3. The position is variable according to your injury **3A**.
- Click in the orange foot fastener on the inside of the foot and retighten the strap with the Velcro fastener.
- Close the click fastener on the lower leg by inserting the orange insertion lug into the plastic shell, then fold it back and click the push button into the plastic shell. If necessary, regulate the tension of the lower leg strap via the upper Velcro fastener **4A**.
- Close the figure 8 strap by crossing it over the top of the foot and leading it backwards towards the calf, where you fasten it with the Velcro ends **5A**.
- Check the fit of the brace. It is important that your ankles are free of pressure and centred by the anatomical side splints on the leg. The positioning of the side splints can be adjusted by the interaction of the front and rear Velcro fasteners. In this case, please consult your orthopaedic and medical supply dealer.

Application of the cooling pad

- Position the cooling pad, with the textile side facing the skin, anywhere inside the brace between the stocking and the brace. We recommend intermittent cooling in the acute phase.

 Store the cooling pad for cooling in the refrigerator only, not in the freezer.

 Please consult your doctor about the use of the cooling pad.

DISMANTLING THE MALLEODYN® S3 IN 3 STEPS

Use the MALLEODYN® S3 immediately after the injury or in the acute phase in the fully rigged state plus cooling pad.

Step 1

Remove the cooling pad of the fully upgraded brace **1B**.

Step 2

Remove the side rail with one click. To do this, press on the orange outer strip and pull the rail out upwards **2B**.

Step 3

Remove the figure 8 strap rein positioned at the bottom of the front edge by loosening the Velcro **3B**.

NOTES FOR THE ORTHOPAEDIC TECHNICIAN

Adjusting the reinforcement rails

- Loosen all fasteners (dorsum of foot, anterior shin fastener, posterior upper and lower fastener).
- Position the foot in the brace. Make sure that the heel rests in the recess provided **1C**.
- Adjust the tension of the anterior and posterior closures so that the lateral malleolus is centred in the designated recess of the lateral reinforcement shell **2C**. The medial plastic splint should also be positioned so that the medial malleolus remains free of pressure.
- First close the foot strap and then the lower leg strap. To do this, click in the orange fasteners and adjust the straps to an optimal tension **3C**.

Adjusting the straps and Velcro fasteners

- If necessary, shorten the protruding Velcro ends on the dorsal closures that you have adjusted to position the lateral reinforcement bars **4C**.
- Shorten the lower leg strap by releasing the lower end from the hook claw, tightening the strap at the lower end and then cutting it flush with the edge of the plastic rail **5C**. The upper Velcro fastener is for fine adjustment by the user.
- Adjust the individual heel width with the Velcro under the sole **6C**.

Dismantling the MALLEODYN® S3

- The reinforcement shell of the lateral splint can be disassembled according to the course of the therapy. To do this, press the orange outer strip and pull the splint out upwards.
- The figure 8 strap can also be disengaged analogous to the course of therapy by loosening the plantar Velcro connection with the brace **7C**.

IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.
- In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at info@sporlastic.de to report the incident.
- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- Product may contain metallic objects. We recommend not putting them in contact with electromagnetic interference.
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

CARE

Before washing, remove the additional side reinforcement rail from the outer plastic rail by unclicking it and removing it upwards. Close all Velcro fasteners.

We recommend washing the product gently in lukewarm water (30°C) with mild detergent and air drying. Do not dry the product in the tumble dryer, on the heating or in the microwave. You can use the laundry bag provided to protect other garments during the washing process.

You can store the disarmed side reinforcement rail and the disarmed figure 8 strap in the laundry bag provided if you want to put them back in.

TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

An online version of the instructions for use can be found on our website at www.sporlastic.de.

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