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67% Polyamid, 33% Elasthan

FUNCTION

The transverse arch of the forefoot is passively restored by circular compression, and the neurovascular bundles (every toe is equipped with two neurovascular bundles) squashed by metatarsal capitula II to IV are protected once again by the arch of the metatarsal capitula 1 B.

INDICATIONS

- Metatarsalgia
- Splay foot
- Periosteal joint irritation of the metatarsal capitula region I – V
- Nerve pressure damage to interdigital nerves II – VIII with
 - burning pain
 - Numbness
 - 'inflammatory' pain
- Cushion-like thickening of the forefoot region with sensitivity to touch and hyperthermia (similar to forefoot arthritis).

CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Severe forefoot blood flow disorders, e.g. diabetes.
- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far no consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

To date, there have been no reports of serious general side effects with proper use and fitting.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restrictive, consistently shaped shaped body fitting.

FITTING

- Place your foot on the bandage, as illustrated in Photo 2.
- The anatomical run of the stitching and the silver-coloured stripes indicate the direction in which the bandage is to be fitted around the forefoot.
- The anatomical stitched shape of the bandage fits the sides of the ball of the big toe perfectly and will not slip. In the further course of events, the inward facing recess indicates the position for fitting to the little toe.
- Next fasten the fastener facing the toes and follow by fastening the second Velcro fastener with the requisite compression pressure.
- The bandage can be worn on the skin or over a stocking. This enables it to be fitted/removed quickly, e.g. when wearing tights.
- The bandage can be worn with the insert provided or with a fitted insole, provided this does not cause you any discomfort.

IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All SPORLASTIC GmbH products undergo product testing as part of our Quality Management System.

However, should you have any complaints about our product, please get in touch with your stockist or directly with us.

CARE

We recommend that you wash the splay foot bandage gently in lukewarm water with cold detergent and air-dry (do not place on radiator).

Close the Velcro fasteners when washing for them to last longer and stop them damaging other items of clothing.

TIP: Slackening tension of the knitted fabric while it is being worn is revived by washing. The bandage should therefore be washed frequently.

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Alle producten van de SPORLASTIC GmbH zijn onderhevig aan de productcontrole van ons kwaliteitsmanagement-systeem.