MITTELHAND-DAUMENSCHIENE

ART.-NR. 07151
MITTELHAND-DAUMENSCHIENE

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INSTRUCTIONS FOR USE EN

Dear Patient,

We ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

INTENDED USE

The MITTELHAND-DAUMENSCHIENE is a thumb brace for immobilisation or partial immobilisation of the metacarpophalangeal and saddle joints of the thumb. The gripping function of the hand is maintained despite immobilisation.

INDICATIONS

- Partial immobilisation of the radial carpus, thumb saddle joint and thumb metacarpophalangeal joint in saddle joint arthrosis
- Arthrosis of the radial carpus and MCP 1
- Ulnar collateral ligament lesion MCP 1 (e.g. ski thumb)
- Navicular pseudarthrosis

CONTRA-INDICATIONS

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be supplied.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenic disorders of the sensory system and skin trophism in the body area to be treated (sensory disorders with or without skin damage).

SIDE EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

PUTTING IT ON AND USE

- The MITTELHAND-DAUMENSCHIENE is available in a left and right version.
- Always slip the affected hand into the padding insert first.
- Now place the MITTELHAND-DAUMENSCHIENE over the padding on the thumb and fix it around the wrist with the Velcro fastener.
- Additional padding inserts are optionally available (ART.-No. 17051).

NOTES FOR THE ORTHOPAEDIC TECHNICIAN

- For an optimal fit, the thumb splint can be thermoformed and professionally customised.

IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.
- In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at info@sporlastic.de to report the incident.
- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.
- Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.
- SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

CARE

We recommend washing the cushion insert gently in lukewarm water (30°C) with mild detergent and air drying. Do not dry the product in the tumble dryer, on the heating or in the microwave. The MITTELHAND-DAUMENSCHIENE can be cleaned with a damp cloth.

TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

An online version of the instructions for use can be found on our website at www.sporlastic.de.

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