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INSTRUCTIONS FOR USE GB

Dear Customer,

we would like to ask you to follow these Instructions for Use with care. If you should have any questions, please consult the doctor treating you, at your nearest stockist or contact us directly.

FUNCTION

The patellar tendon bandage integrated in the silicone ring 1 of the SUPER-GENUPLUS knee bandage ensures faster and stronger activation of the femoral musculature.

Just as a tap with the reflex hammer on the patellar tendon results in a muscular reaction, so does the constant pressure of a bandage on this point result, by way of the reflex, in a permanent improvement of muscular activity. Stability within the joint is increased whereas pain is reduced; the joint is protected.

This active muscular reaction can be specifically exploited with frontal knee pathologies, supplemented by physiotherapeutic exercises, and prophylactic with sports injuries.

As a result of intensified friction, the burred surface of the silicone ring generates, by way of the skin, a more intensive response of the muscular-activating sensors. The additional compression effect of the knitted bandage results in an improvement of blood circulation with a positive effect in connection with insertion endopathy and effusion resorption.

INDICATIONS

- _ Frontal knee pain
- _ Patellar tip syndrome
- _ Chondropathia patellae
- _ Femoral patellar arthrosis
- _ Cruciate ligament insufficiency
- _ Frontal cruciate ligament
- _ Post-operative / post-traumatic rehabilitation training: Improvement by way of intensified, yet balanced, neuro muscular recruitment
- _ Quadriceps paresis
- _ Muscular dystrophy
- _ Pan-gonarthrosis with effusions
- _ Chondromalacia patellae and gonarthrosis with insert endopathies
- _ Inflammatory rheumatic knee joints

CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- _ Post-thrombotic syndrome.
- _ Long-lasting, rigid immobilisation, especially in the elderly.
- _ Allergic, inflammatory or lesion-specific skin alterations

(e.g. swelling, reddening) of the body areas being treated.

- _ Impaired circulation or lymphatic soft tissue swelling.
- _ Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far no consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

To date, there have been no reports of serious general side effects with proper use and fitting.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restrictive, consistently shaped shaped body fitting.

FITTING

- _ Pull the knee bandage over the knee in such a manner that the knee-cap is seated in the recessed part of the silicone-friction truss pad.
- _ The lower part of the truss pad will then be automatically located on the patellar ligament.
- _ Lightly pull the side straps and close them over the hollow of the knee with the Velcro fastener.

IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

CARE

Wash SUPER-GENUPLUS in lukewarm water with a cold-water detergent and air dry. Do not place on a radiator. Close Velcro fasteners before washing as this will lengthen their useful life and avoid damage to other laundry.

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