SPORLASTIC ORTHOPAEDICS

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SPORLASTIC GmbH

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Weberstraße 1 _ 72622 Nürtingen _ Germany

Postfach 14 48 _ 72604 Nürtingen _ Germany

Telefon +49 70 22/70 51 81 _ Fax +49 70 22/70 51 13

info@sporlastic.de _ www.sporlastic.de

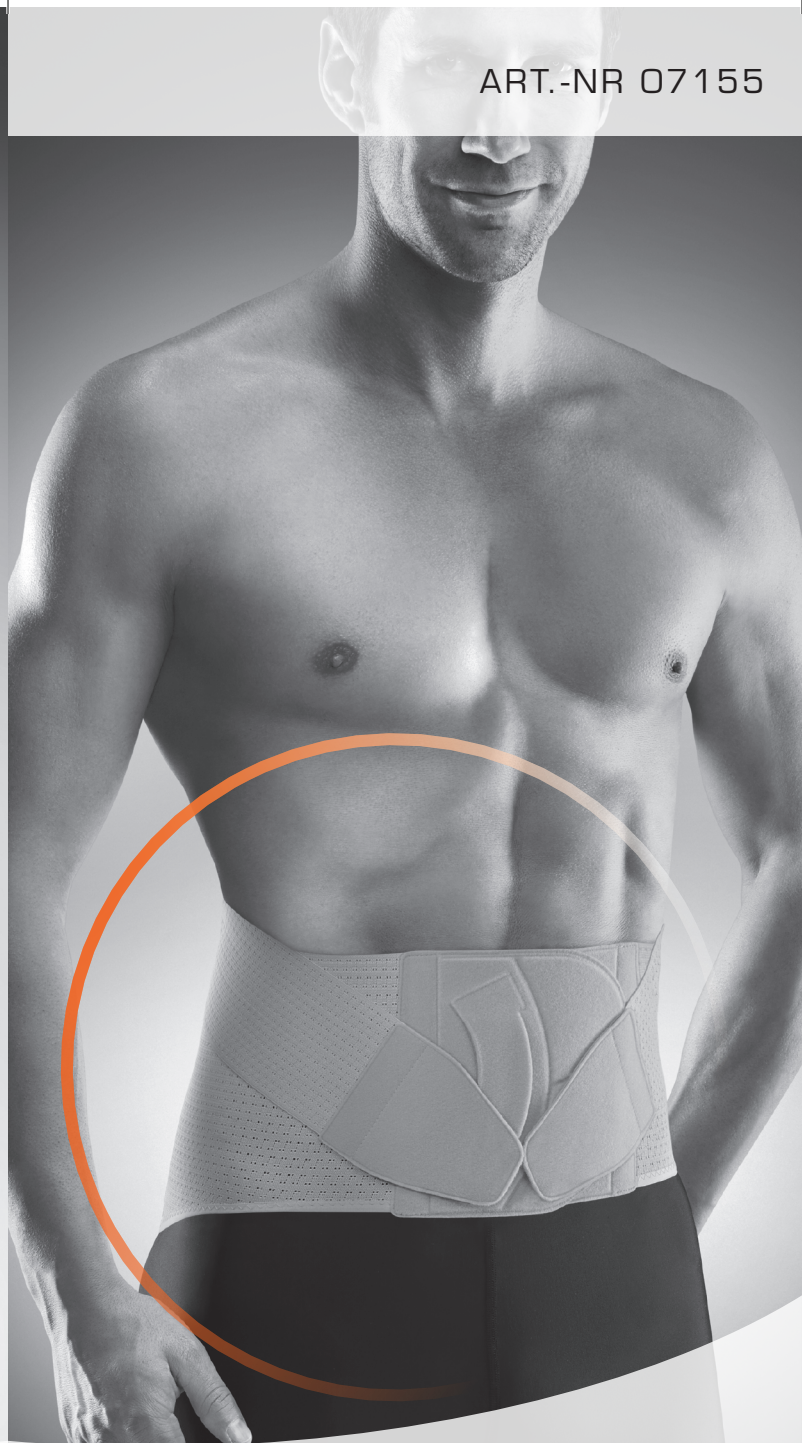
CE

 **SPORLASTIC®**

ART.-NR 07155

VERTEBRADYN®-LIGHT
SPORLASTIC®

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INSTRUCTIONS FOR USE GB

FUNCTION

VERTEBRADYN®-LIGHT is an orthotic device for stabilization of the lumbar vertebral column (lumbar spine) with a relieving effect.

The anatomically pre-formed spring steel rods integrated into the back of the device together with the pressure which develops via compression in the abdomen stabilize and bring relief to the lumbar spine and the transition between the lumbar spine and the sacrum. Using the continuously adjustable Velcro closure, the compression pressure of the VERTEBRADYN®-LIGHT and the degree of stabilization can be individually adjusted using the two elastic tension elements (tension straps).

By using the hand loops attached to the closure, the orthotic device is particularly convenient to put on.

The anatomically designed orthotic device made of elasticated and breathable material provides functional compression with high wearing comfort.

INDICATIONS

- _ Lumbar spinal syndrome
- _ Slight instability of the spinal column
- _ Spinal disc damage
- _ Preventative stabilization

CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- _ Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- _ Impaired circulation or lymphatic soft tissue swelling.
- _ Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far not consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

With proper use and proper fitting, so far there have been no reports of serious general side-effects.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restricting consistently shaped body fitting.

FITTING

1 _ Place the orthotic device around the abdomen. The tension straps are located at the top and the stiffening rods are positioned in the middle of the back. Put your fingers into the loops attached on both sides of the closure of the VERTEBRADYN®-LIGHT - pull apart the orthotic device under tension and bring the closure evenly to the front.

2/3 _ Firstly, place the left closing half on the abdomen and pull the right closing half as far as is possible to the front until fastening the hook-and-loop closure is possible. Now pull your fingers out of the loops and close the Velcro fastener of the VERTEBRADYN®-LIGHT completely. When making the hook-and-loop closure, ensure that the closure plates are placed evenly over each other.

TIP: Wear the VERTEBRADYN®-LIGHT over a tight-fitting T-Shirt or undershirt.

4 _ Pull the right and left tension straps forward and slightly down to create an even tension.

TIP: The VERTEBRADYN®-LIGHT should have a tight fit on the body in order to have sufficient compression. This can be achieved through the combination of the stabilizing elements in the lumbar region, the tensioning straps and the abdominal support in order to give partial relief and stabilization of the lumbar spine. When putting on the high compression device, make sure it is comfortable to wear.

5 _ Attach the ends of the tension straps under tension to the lower edge of the base closure. Ensure that the Velcro surfaces are placed completely over each other and that contact with clothing or skin is avoided.

TIP: The front lower end of the VERTEBRADYN®-LIGHT should sit flush to the thighs when seated and thereby exert no pressure on the groin.

NOTE FOR THE ORTHOPAEDIC TECHNICIAN

The spring steel rods can be adapted to match the individual anatomy, if necessary. They cannot be removed.

IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

CARE

We recommend to hand wash the orthotic device in lukewarm water with a mild detergent, allowing it to dry in the air and not to put it on the radiator. In order to avoid damage to other items of clothing during washing, the orthotic device is best washed separately - close the hook-and-loop closure beforehand.

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