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INSTRUCTIONS FOR USE EN

INTENDED USE

VERTEBRADYN® STRONG is a lumbar support brace for partial relief by de-lordosis of the lumbar spine.

INDICATIONS

- Severe lumbalgia
- Muscular imbalances
- Irritations of the sacroiliac joint
- Facet syndrome
- Spinal canal stenosis
- Spondylolisthesis
- Herniated disc
- Instability of the spine
- Lumbar deformity

CONTRA-INDICATIONS

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be supplied.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenic disorders of the sensory system and skin trophism in the body area to be treated (sensory disorders with or without skin damage).

SIDE-EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

PUTTING IT ON AND USE

- We recommend wearing a thin, tight-fitting T-shirt under the product.
 - The VERTEBRADYN® STRONG is correctly positioned when the back section with the spring steel rods lies centrally over the spine and ends with the orange-coloured lower edge over your sacrum **1**.
 - To close the lumbar support brace, insert your fingers into the hand loops attached to both sides of the closure. Pull the brace apart behind your back with tension and guide it around your body. Tighten the two fastening plates under tension in the middle of your abdomen **2**.
 - Pull your fingers out of the loops and close the Velcro fastener of VERTEBRADYN® STRONG completely. When Velcroing, make sure that the closure plates lie positively on top of each other **3**.
 - Pull the lower right and left traction elements forward evenly with tension and upwards at a slight angle. Attach the ends of the lower traction elements to the locking plates under tension.
 - Pull the two upper tension elements forward evenly with tension and slightly diagonally downwards. Also attach the ends of the upper traction elements to the locking plates under tension **5**.
- VERTEBRADYN® STRONG should fit snugly with high compression. In this way, the combination of the abdominal support with the straight, lumbar spine-bridging spring steel rods and the tension elements can provide significant relief for the lumbar spine. When putting on with high compression, make sure that it is comfortable to wear.
- The front lower end of VERTEBRADYN® STRONG should be flush and parallel to the thigh when seated and should not cause pressure in the groin **6**.

NOTES FOR THE ORTHOPAEDIC TECHNICIAN

- The spring steel rods can be adapted to the individual anatomical conditions.

ACCESSORIES

- The BAUCHPLATTE FÜR RÜCKENORTHESSEN can be ordered as an option for the product. It can surround prominent abdominal structures over a large area and thus support the weight of the abdomen (Art.no. 17428).
- Optionally, the VERLÄNGERUNGSTEIL RÜCKEN can be ordered with the product. It is attached ventrally to the closure plates and serves as a closure extension for larger circumferences (Art.no. 17400).

⚠ IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.

- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.
- In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at info@sporlastic.de to report the incident.
- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- Product may contain metallic objects. We recommend not putting them in contact with electromagnetic interference.
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

CARE

We recommend washing the brace gently in lukewarm water (30°C) with mild detergent and air drying. Do not dry the product in the tumble dryer, on the heater or in the microwave. To avoid damaging other items of clothing during washing, it is best to wash the brace separately and close the Velcro fasteners beforehand.

Any tension in the knitted fabric that decreases over time is restored by washing. We therefore recommend washing the brace more frequently.

TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

MATERIAL COMPOSITION

67% polyamide, 33% spandex

An online version of the instructions for use can be found on our website at www.sporlastic.de.

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